CHARISMA® Diamond

CHARISMA® Diamond
6898526703

- Instructions for use
- Instrucciones de uso
- Instruções de uso
- تعليمات الاستخدام
- 사용 설명서
- 使用說明

Dentistry – Polymer-based restorative materials
ISO 4049
(type 1, class 2, group 1)

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Kulzer South América Ltda.
CNPJ #48.708.010/0001-02
Rua Cenno Sbrighi, 27 - cj. 42
São Paulo - SP - CEP 05036-010
sac@kulzer-dental.com

Resp. Técnica: Dra. Regiane Marton – CRO 70.705
Nº ANVISA: 10166840079

Manufacturer:
Kulzer GmbH
Leipziger Straße 2
63450 Hanau (Germany)
Made in Germany

CE 0197

Explanation of symbols on labelling	
MD	Medical device
LDT	Batch code
■	Manufacturer
□	Date of manufacture
REF	Reorder no.
Keep away from sunlight / radioactive sources	
Consult instructions for use	
Use-by date	
Storage temperature	
Contains nano materials	
UDI	Unique Device Identification

CHARISMA® Diamond

Instructions for use (GB)

Medical Device – for use by dental health care professionals only.

Intended Purpose

CHARISMA Diamond is a light curing, radio-opaque nano composite. It is indicated for adhesively bonded, tooth coloured restorations of anterior and posterior teeth including occlusal surfaces. Due to the excellent material properties, a aesthetic result is durable, high lustre polish is easily reached. Its radiopacity is equivalent to 2.5–3 mm of aluminum (dentin = 1 mm, enamel = 2 mm).

This dental material is suitable for patients requiring dental treatment for the following indications with consideration of the contraindications. For pregnant and nursing women, the treatment risks must be weighed carefully against the benefits taking into consideration the unborn child or infant.

Kulzer medical devices ensure the rehabilitation of oral functions such as chewing, speaking and aesthetics. They stabilize the remaining dentition and/or the alveolar ridge restoratively or prosthetically.

Composition

Benzocaine/Bromo Fluor Silicate Glass, TCD-Urethaneacrylate, Silica, UDMA, TEGDMA, Titanium Dioxide, Fluorescent Pigments, Metallic Oxide Pigments, Organic Pigments, Aminobenzoic-acid, BHT, Camphorquinone. CHARISMA Diamond contains approximately 64% filler by volume, its filler particle size is 5 nm – 20 µm.

Clinical Indication

Application to close the pulp (less than 1 mm). The use of this product is contraindicated in case of known or suspected allergies against components of this product. Do not use this product in case of known or suspected allergies against metacrylic acid composites.

Potential side effects

This product or one of its components may in particular cases cause hypersensitivity reactions.

Warnings and safety instructions

May cause skin and eye irritation. Avoid contact with skin. Skin contact may cause sensitization. Wear suitable protective gloves. In case of contact with skin, immediately wash with plenty of water. Avoid contact with eyes. In case of contact with eyes, rinse with water for several minutes. If symptoms persist, consult an eye specialist.

Application hints

Visually inspect for damage before use. Damaged products must not be used. Composites should only be used at room temperature. In the case of deep cavity preparations in close proximity to the pulp or a direct pulpal exposure, cover the dentine with a calcium hydroxide liner covered with a glass ionomer cement before applying the adhesive. Do not use eugenol containing materials in or around the preparation being restored, because this may interfere with the proper curing of the composite. Use adequate protection for syringes to prevent from any contamination. Do not re-use after contamination.

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